





CryoPen sets a new standard for the cryogenic treatment of benign skin lesions, offering the ultimate in control, convenience, and cost-effectiveness - as well as an outstanding patient experience.

This high-performance, ergonomic cryotherapy pen dispenses a fine jet of nitrous oxide under high pressure, allowing you to work with millimeter precision.

The CryoPen O+ is an all-purpose model, suitable for regular daily use, and can take two different sizes of cartridge.

Key features

- Ergonomic design for superior control
- Leading-edge microtechnology ensures controlled gas flow
- Takes easy-to-load 8g or 16g N₂O cartridges
- Aluminum body with applicators in medical-grade materials
- Provided with 4 autoclavable applicators
- Suitable for regular use
- Certified (€ 1639 FDA




















APPLICATORS

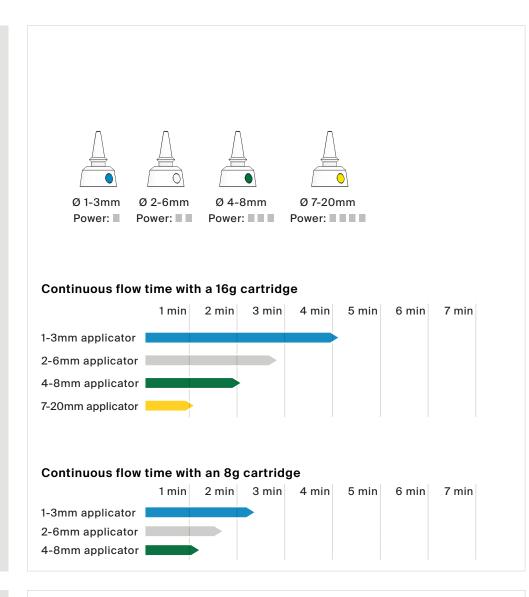

PRODUCT INFORMATION

Device

The CryoPen O+ is suitable for all treatment situations, whether more delicate or stronger freezing power is required. The device can be used with 8g or 16g cartridges, and with the complete range of applicators, making it a highly versatile instrument.

What kind of lesions?

Acrochordon, actinic keratosis and cheilitis, capillary hemangioma, cutaneous leishmaniasis, discoid lupus erythematosus, filiform warts, flat warts, vulgar warts, plantar warts, anogenital warts, rosacea, lichen sclerosus, granuloma annulare, molluscum contagiosum, seborrheic keratosis and viral lesions.

Procedure

No special preparation is required. Apply the jet of nitrous oxide for 2 to 45 seconds on each lesion, depending on the size and depth of the lesion. For larger lesions, use circular or back-and-forth movements to treat all the tissue.

PRODUCTSHEET_CryoPenO+_MED_20230412

info@ho-equipments.com | Tel. +32 68 26 86 00 | www.ho-equipments.com







Device

CryoPen O+

Applicators

1-3mm applicator

2-6mm applicator

4-8mm applicator

7-20mm applicator

Cartridges

6x 16g cartridges

Accessories

Metal carrying case

Forceps

Converter for 8g cartridge

Device stand

Documentation

Treatment overview User manual

Dimensions

Device: 22.3 x 3.3 x 3.3 cm

Carrying case: 28.8 x 22.8 x 6.7 cm

Starter set: 38 x 30 x 7.5 cm

Weight

Device with a full cartridge: 300g

Starter set: 2,470g

HS codes

90189084 (device) 28112930 (cartridges)



info@ho-equipments.com | Tel. +32 68 26 86 00 | www.ho-equipments.com